**WARNING:**

- Always use these hand-grips to control and drive the column and arm movements, never push directly on the X-ray tube or collimator.
- Rotate the column and move the telescopic arm after releasing the brake. Do NOT rotate the column or move the telescopic arm by force without releasing the brake. The locking system can be damaged if an attempt is made to move the column or telescopic arm with force.
- Be careful not to get your hand caught when operating the column arm.

Movement Controls of the Column and Telescopic Arm 4.3.1

The column can rotate through $\pm 315^\circ$ from its parking position. The arm allows a vertical travel of 1,480 mm and a telescopic horizontal travel of 562 mm.

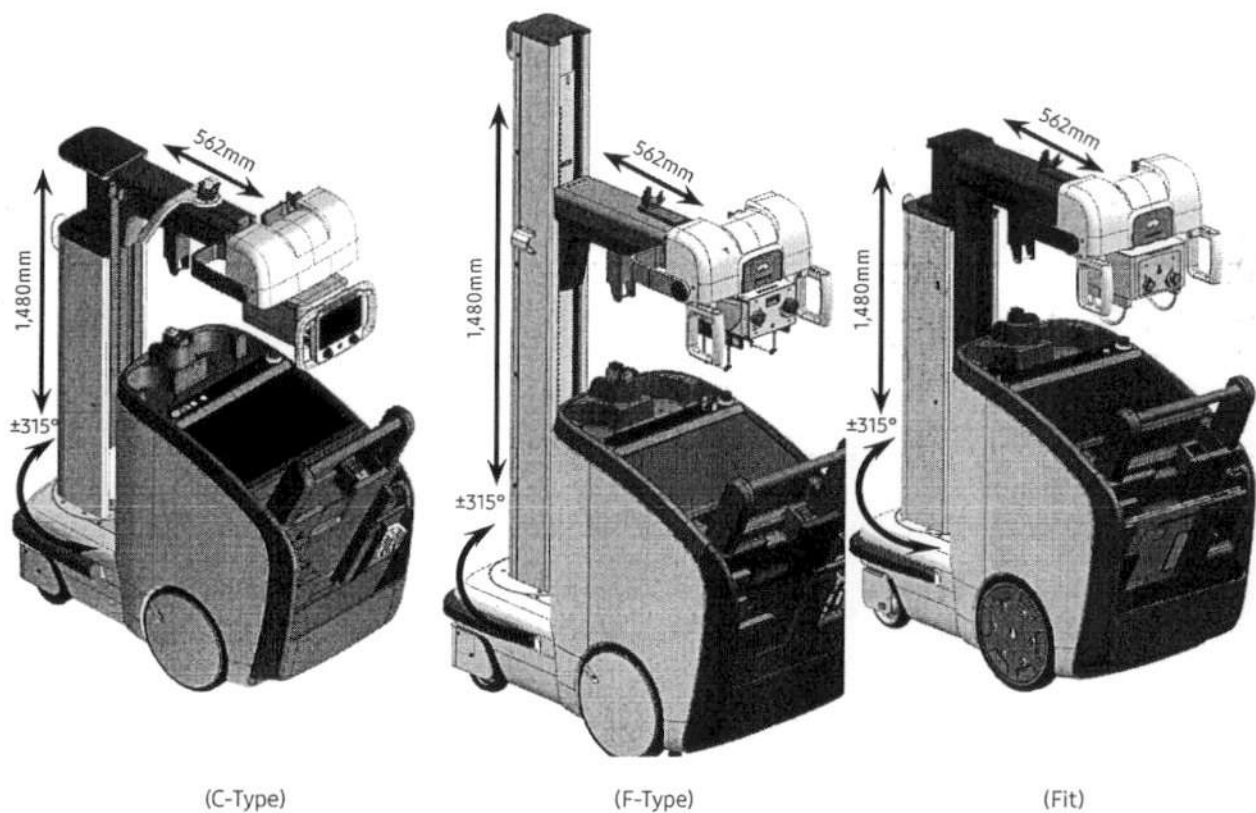
Figure 3-35. Column and Arm Movement

Setting the Capturing Conditions

You can check or set the parameter information required for capturing images.

4.1.5

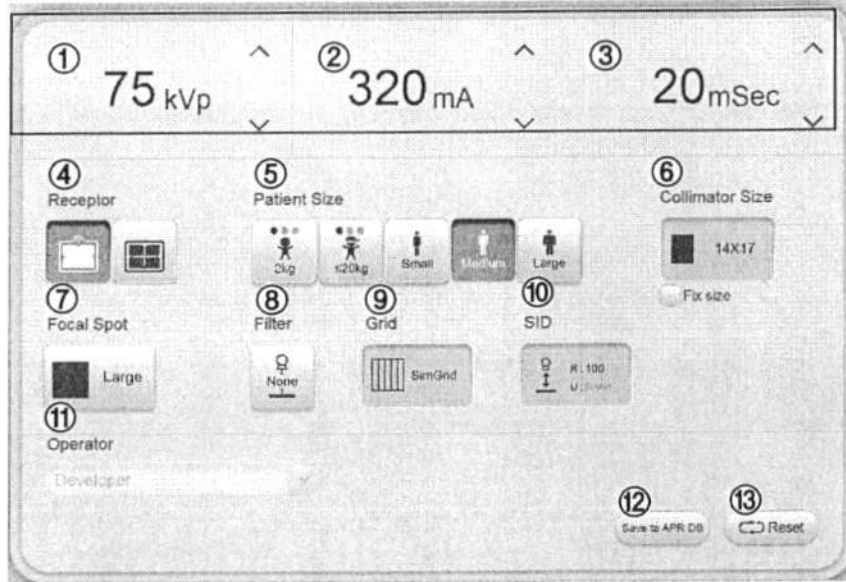
Figure 4-10. Set the capturing conditions screen

No.	Name	Description
①	Tube voltage	Displays the tube voltage. The value can be set with the \wedge button and \vee button.
②	Tube current	Displays the tube current. The value can be set with the \wedge button and \vee button.
③	Irradiation Time	Displays the mSec or mAs value. The value can be set with the \wedge button and \vee button.
④	Receptor	Shows the current information about the selected receptor. You can choose a different receptor. : Portable Detector : There is more than one portable Detector. : Cassette Capturing * If you use multiple detectors, on the screen select the icon whose color matches the LED color icon of the detector you want to use. This will activate the detector and display the size of the detector.

Handwritten signature

5.3

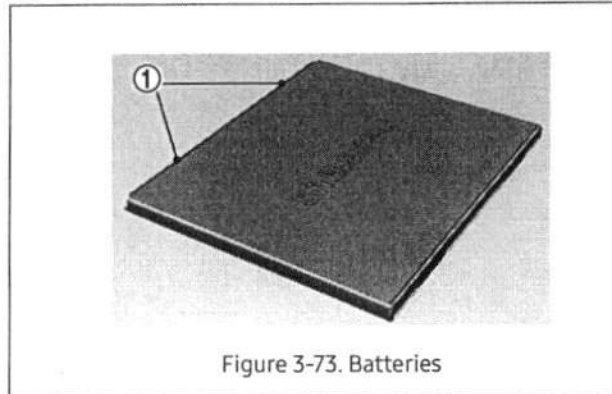
Figure 3-73. Batteries

① Battery Terminal

- ☑ • If you notice that the battery life has become much shorter than normal, replace the battery. The battery is a consumable and has a warranty period of 6 months.
- If you are not going to use the battery for a long time, fully charge the battery, and then remove it.

CAUTION: Use caution as mishandling the Li-ion battery may cause heat, fire, explosion, and/or deterioration in performance.

- If you are not going to use the battery for a long time, remove the battery. Then, store the battery in a dry and cool place.
- Avoid using the battery in places where static electricity, which may affect the battery protection elements, can occur.
- Keep the battery away from direct sunlight and high temperatures.

5.4

■ Battery Charger (Option)

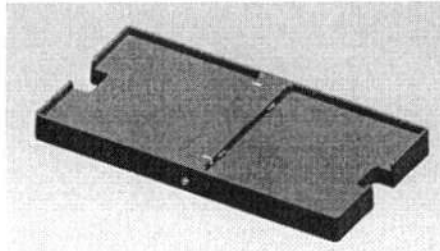
Figure 3-74. Battery Charger

This device is used to charge the detector battery.

- Battery Installation

Slightly tilt the top part of the battery and insert the opposite part into the battery slot until its terminal touches the charger's connector. Then, gently press down on the battery to ensure that it sits firmly in the slot.

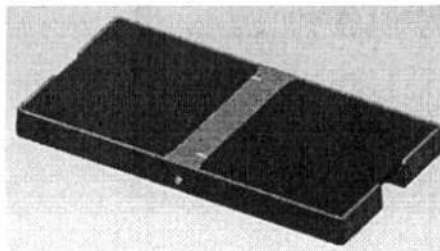
Figure 3-75. Battery Insertion

- ☒ • The battery is a consumable and has a warranty period of 6 months.
- The battery charger is used only with the wireless detector.

Bluetooth Type 4.1.6

Take the Bluetooth type wireless exposure switch out of its cradle. Press the Ready/Exposure button in sequence for wireless exposure. Wireless working distance is maximum 10 m. It can be operated in the same way as the 3 button-wired exposure switch.

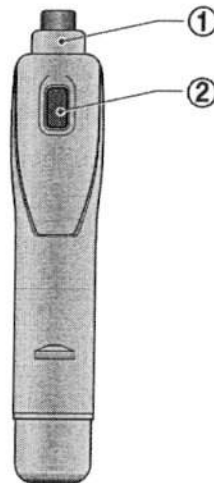
Figure 3-44. Wireless Exposure Switch (Bluetooth Type)

- ① Ready/Exposure Button
- ② Collimator Lamp Button

Follow the steps below to operate the bluetooth type wireless exposure switch (same as wired exposure switch button 3).

- Take the wireless exposure switch out of its cradle.
- The wireless exposure switch turns to preparation status when the Ready button is pressed and turns to irradiation status when Exposure button is pressed while pressing the Ready button. Press the Exposure button until the exposure is completed.
- When the exposure is completed, return the switch back to its cradle.

- ☑ • Infrared or Bluetooth type wireless exposure switches can be powered off when Wireless Switch Power is set OFF on the configuration menu of S-Station (Setup > General > Etc). Ensure that exposure is not allowed and wireless exposure switch is not available when OFF is selected. Always ON has to be selected when an Infrared or Bluetooth type wireless exposure switch needs to be operated.
- The Bluetooth type wireless exposure switch may not be available in some countries due to pending regulatory approval.
- Communication error in wireless bluetooth connection can be caused when:
 - There are wireless devices with 2.4 GHz nearby.
 - There are devices such as microwave oven, equipment for hyperthermal therapy that use microwave.
 - There is an obstacle between hand switch and holder.

10591

4.1.6

■ **Wired Exposure Switch (Button 2/Button 3 (Option))**

The standard wired exposure switch has 2 buttons and the optional wired exposure switch has 3 buttons.

Wired Exposure Switch (Button 2)

Figure 3-40. Wired Exposure Switch (Button 2)

① Ready/Exposure Button

Handwritten signatures and initials in black ink, including a large signature that appears to be 'วิไล' and some smaller initials below it.

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(Ms. APIRADA PUAPORNONG)

Associate Director - Head of Display Group

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This certificate validity period is 1 year.

Certificate of Online Training

36

SAMSUNG HEALTHCARE TRAINING CENTER certifies that

attapon 6.3 7.3

of J.F. ADVANCE MED CO.LTD.

Has successfully completed all course requirements for

GM85 Basic Online Training

Online On Apr 30, 2021 ~ Jul 14, 2021

This course contains technology of SAMSUNG product,

GM85

SAMSUNG

Jeonhyeok Cho
Director
Service Solution Group

SHMEON-DR-210432

Certificate of Application Training^{6.6}

Samsung Electronics Certifies That

J.F. Advance Med / Chutimon Sanjai

has Successfully Completed All Courses Requirements For

2023 DR Application Training

In Seoul, Korea On March 21-23, 2023

This Course Contained Technologies of Samsung Electronics Products

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